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## Really Locked!

What is the use of a “seal” on a closure when the complete container including the barrier and the gasket is not protected against undetected opening?

SteriSet containers, with the patented **ThermoLoc closure**, provide a solution to this required sealing in an outstandingly elegant manner: **They “seal” themselves during sterilization**. By means of the temperature effect, a color-marked catch is slid in front of the closure latch such that swinging it open is not possible. This “sealed” state is visually recognizable – and provides above all a **perceptible blockage of the closure**: The barrier cover together with the gasket is then securely “locked” with the lower part – and reusable without handling and without consumables.

The container is opened in **two** steps:

- 1 Intentional unsealing of the closure blockage,
- 2 Swinging open of the closure latches.

This eliminates the meticulous visual checking of both container fronts required by other systems for intactness of the sealing, which is an invaluable advantage in the often hectic operating room routine.

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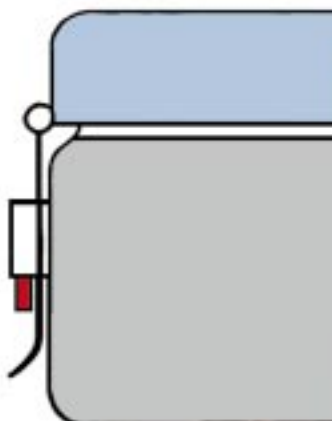
The latch of the outer cover is securely blocked against being swung open by the stable, colored catch of the ThermoLoc closure.

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The ThermoLoc closure in sealed and unsealed condition.

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ThermoLoc containers are unsealed after cooling by a simple hand motion (upward pressing) on the colored catch. In this manner, the closure is also automatically prepared for the next sealing (sterilization).

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